

## **Technical Alert**

## Discontinuation of Hemoccult Fecal Occult Blood Test due to Long Term Supply Disruption

**TO:** Medical Staff and Clients

**FROM:** Dr. A. Christian Whelen, PhD, D(ABMM)

V.P. - Technical Director (Microbiology)

Terrie Koyamatsu Dr. Wesley Kim, MD Dr. Ana Ortega-Lopez, MD

Manager, DLS Microbiology Medical Director DLS Medical Director QMC Punchbowl,

and QMC West North Hawaii and Molokai

**DATE:** April 8, 2021

SUBJECT: EFFECTIVE IMMEDIATELY - Discontinuation of Hemoccult Fecal Occult Blood Test

The distributor for Hemoccult has not filled supply orders for Diagnostic Laboratory Services, Inc. since November, 2020. Despite repeated inquires, they will not commit to an allocation or provide any delivery timeframe. Our testing supplies have run out, and we are having to cancel orders as specimen stability limits are exceeded. We cannot locate another laboratory who will accept these specimens. We are forced to discontinue this test. Please **do not give patients Hemoccult cards** for specimen collection – we have no way to test them.

The Quidel QuickVue has been an option for testing at DLS for several years. It uses similar immunochemical technology to detect human hemoglobin (hHb), has an analytical sensitivity of 50 ng hHb / ml, and only requires 1 specimen. In contrast, Hemoccult is less sensitive at 300ng hHb / ml, so requires 3 collections

**Please immediately discard any remaining stock of Hemoccult cards** and order QuickVue collection tubes from DLS Purchasing.

We apologize for any inconvenience this has caused you or your patients. We sincerely believed the supply disruption would be resolved in a timely manner, which it was not.

Test	Specimen	Unit Code	Collection	Turn-around Time
Occult Blood,	iFOB Collection Tube	602	Daily	<24 hours from receipt
Stool, iFOB				of sample

Specimen stability in iFOB Collection Tube is 8 days ambient and 6 months at 2-8°C (refrigerated).

Please refer any questions to Terrie Koyamatsu, Manager - DLS Microbiology Laboratory at 589-5196, or DLS Client Services at 589-5101.